

Steps for Disinfecting Telehealth Equipment and Peripheral Devices



1

PERFORM HAND HYGIENE

A



B



Perform hand hygiene using an alcohol-based hand rub (A) or soap and water (B).

Touch panels, remote controls and peripherals will be cleaned before and after each use with these four steps.

2

USE READY-TO-USE LOW LEVEL DISINFECTANT



Remove wipe, close lid tightly and ensure to squeeze wipe to remove excess fluid. Follow manufacturer label for glove use.

3

DISINFECT



Use one side of the wipe to clean the surface. Use the other side of the wipe to disinfect, discard. Allow surface to fully air dry.

4

PERFORM HAND HYGIENE

A



B

