

Varch mask adjustments, donning and doffing

1




Clean hands before taking mask out of box.

2




Take mask out of the box. Pull open the pleats slightly, and keep the blue side facing outward, away from you.

3

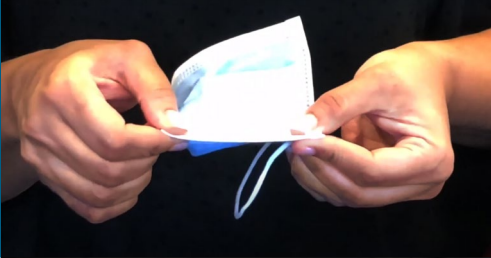

Fold mask in half and give it a firm pinch along the fold. The blue side is always facing out.

4

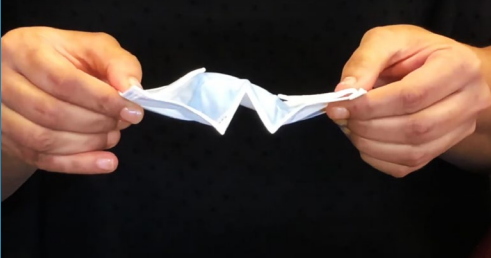

Fold it again, making a W shape along one horizontal side; this will become the top of your mask, where it fits your nose.

5



Put on mask and position it over mouth AND nose.

6



To doff your mask, clean hands again.

7



Then pull it off from behind your ears and drop it straight into the garbage.

8



Complete your hand hygiene routine again, after removing your mask.