






- Notes**
- This is the preferred specimen for *Chlamydia trachomatis*/ *Neisseria gonorrhoeae*/ *Trichomonas vaginalis* testing in females
 - Only one specimen collection is required to perform one or all tests for *Chlamydia trachomatis*, *Neisseria gonorrhoeae*, and *Trichomonas vaginalis* nucleic acid amplification testing (NAAT)
- Risks or Cautions**
- **Use only the pink swab that comes with the kit**
 - **Do not** place endocervical, or urethral swabs in this container
 - If the soft tip of the swab is touched, laid down on a surface or dropped, discard and obtain a new collection kit
 - If the contents of the transport tube are spilled at any time during the collection procedure, use a new APTIMA® Multitest Swab Specimen Collection Kit. Failure to use a new kit may invalidate the test results.

Materials Required APTIMA® Multitest Swab Specimen Collection Kit

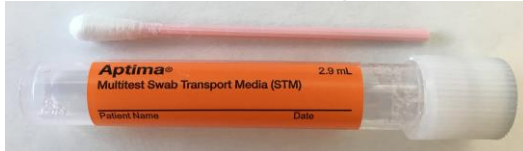

Collecting the Sample

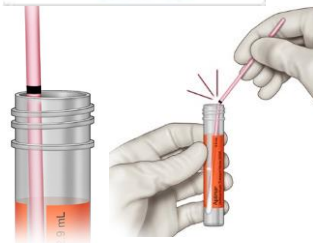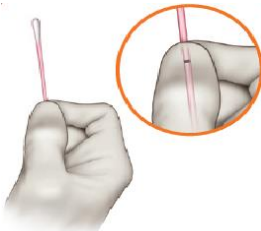

- Partially peel open the swab package and remove the swab.
- **Do not** touch the soft tip or lay the swab down
- **Hold the swab, placing the thumb and forefinger in the middle of the swab shaft covering the black score line.** Do not hold the swab shaft below the score line during collection.
- Carefully insert swab into vagina about 2 inches (5 cm) past the introitus and gently **rotate the swab for 10-30 seconds**
- Ensure the swab touches the vaginal walls so that the moisture is absorbed by the swab
- Withdraw swab without touching the skin
- While holding the swab in the same hand, unscrew the tube cap
- Immediately place swab in transport tube so that the tip of the swab is visible below the tube label (immersed in transport media) and the score line is at the top of the tube
- Break swab shaft at the score line against the side of the tube. **Do not** touch the inside of the tube.
- Discard top portion of the swab shaft
- Tightly screw cap onto tube

Labelling the Sample

Label the collection container with:

- The patient **full first and last name, date and time of collection, Provincial Health Number (PHN/ULI)**, or Government issued identification such as Federal, Military, RCMP, or Immigration.
- If a patient demographic label has been provided, attach this label to the outside of the container

Lab Requisition

- Print the date and time the sample was collected on your lab requisition
- **Fold** and place the requisition in the **outside** pocket of the plastic resealable bag.

Sample Delivery to Lab

- Vaginal swab specimens must be transported to Alberta Precision Labs in the provided swab specimen transport medium and tube within 72 hours of collection
- Store at room temperature

Source images courtesy of HOLOGIC, Inc, Aptima® Multitest Swab Specimen Collection Kit Clinician Collection Procedure Guide for Vaginal Swab Specimens DS-01552-001 Rev.006 2020

Questions? Contact Alberta Precision Laboratories at 403-770-3600