

***** IMPORTANT *****

Aseptic techniques must be used at all times.

**Recommended Guidelines for Reconstitution in the Event of Loss of Vacuum with
the Mix2Vial™**

Supplies Required:

Appropriate size syringe (10 or 20 cc)
2 antiseptic swabs

1. Grasp the outer rim of each of the Mix2Vial™ components.
2. Unscrew counterclockwise to separate.
3. Place both vials on a clean surface or countertop.
4. Carefully detach blue component of the Mix2Vial™ from the diluent vial and discard.
5. Swab the diluent vial top with an antiseptic swab.
6. Without touching the white luer lock tip, attach a clean syringe to this end, tightening firmly. Remove clear component of Mix2Vial™ from product vial (with attached syringe) while maintaining aseptic technique.
7. Wipe product vial top with an antiseptic swab.
8. Draw a volume of air into the syringe that is equivalent to the remaining volume of diluent in the vial.
9. Place the clear component with the attached syringe into the diluent vial, and inject the air into the diluent vial.
10. Invert vial and draw remaining diluent into syringe.
11. Remove clear component of the Mix2Vial™ with attached syringe from the diluent vial.
12. Place the clear component with the attached syringe containing the diluent into the product vial.
13. Carefully push diluent into product vial and proceed as usual.

*Mix2Vial™ is a trademark of West or one of its subsidiaries.

COMMENTS:

Date Effective: Jan. 2013

Revised Date: 18 Jan 2013

Version: 1.1

Approved By: TM Integration Network

For questions or comments, please contact trevor.richardson@albertahealthservices.ca