

How to Label and Mix an Excyte ESR Vacuum Tube

This document is applicable to all personnel at site(s):

All Sites

Procedure


This document provides instruction on how to label and mix an Excyte ESR vacuum tube.

Specimen Labeling Information

Specimens **must** be labeled at the patient bedside immediately after collection. This will help prevent mislabeling of specimens with the wrong patient information.

A properly labeled specimen contains the following information:

- ✓ Full First and Last Name
- ✓ PHN/ULI
- ✓ Collection Date and Time
- ✓ Your Initials

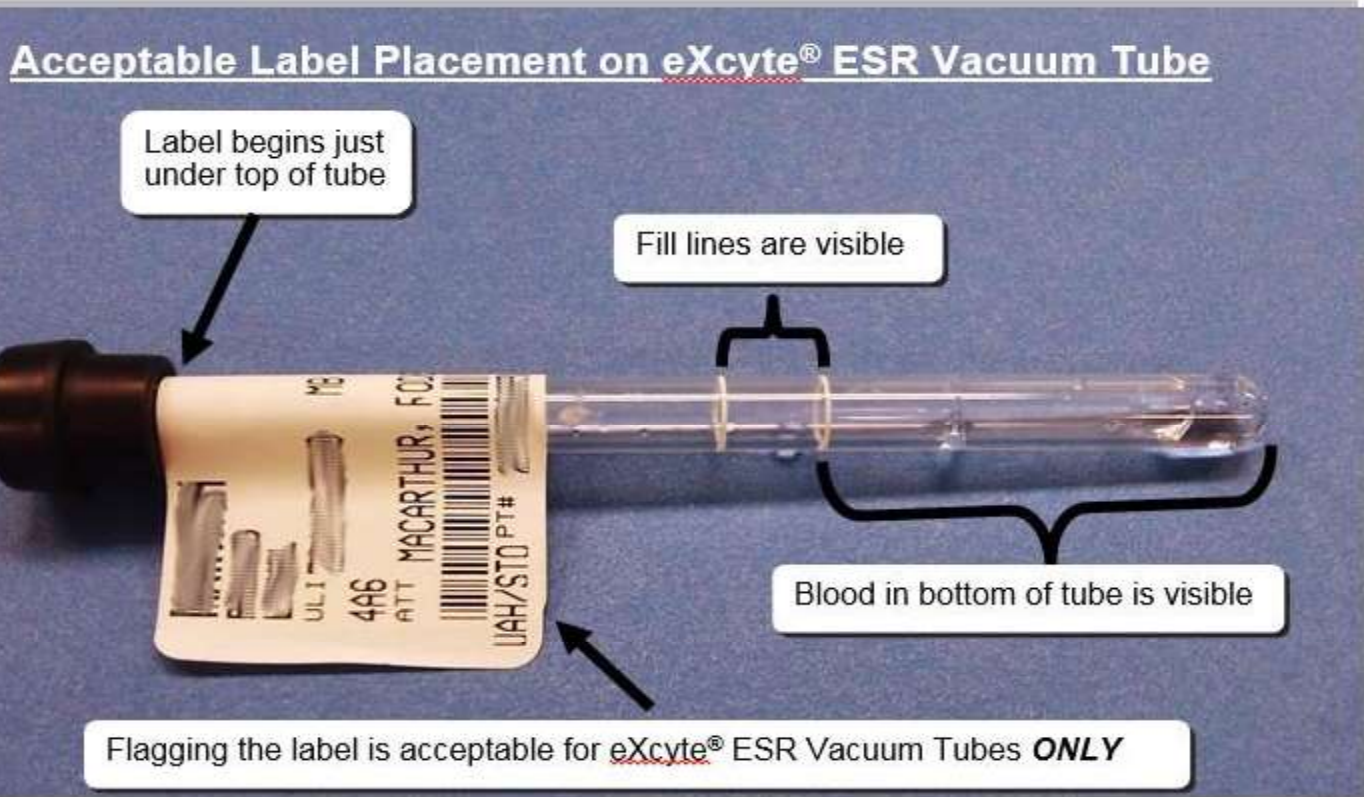

**Mix the tube with gentle inversions 10 times.
Ensure the blood flows to the top and bottom with each inversion.**

Contact Information

University of Alberta Hospital Client Response: 780-407-7484.